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REF G-37551

**ILM-SCRAPER**

**PRODUCT RELATED PROCESSING INSTRUCTIONS**

These product related processing instructions are for critical\* medical devices (MD) are exclusively valid for the above listed items by GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

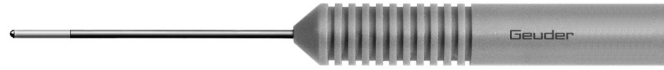
Fig. 1: ILM-Scraper, not to scale

**OVERVIEW**

Cycle limit	Assembly/Disassembly	Accessories
25 cycles	NO	NO
Pre-cleaning	Ultrasound	Automated processing
YES	YES	YES
Manual processing	Compressed air	Steam sterilization
NO	YES	YES

Tab. 1: Overview

**AUXILIARY EQUIPMENT**

<b>First treatment</b>	Soft tissue (e.g. compress)
<b>Pre-cleaning</b>	Soft plastic brush or steamer (steam jet, max. 2 bar), single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar) optionally: ultrasound bath 35-40 kHz
<b>Drying</b>	Medical compressed air (oil-free compressed air, max. 2 bar)
<b>Checking pH value</b>	pH test strips

Tab. 2: Auxiliary Equipment

**FIRST TREATMENT AT PLACE OF USE**

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/ immediately after application with a compress soaked in sterile water.
- Place dry ILM-Scraper into a suitable storage or transport system and close it.

**PREPARATION BEFORE CLEANING**

**Pre-cleaning**

- Rinse the ILM-Scraper under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.

**OR**

**Ultrasound cleaning as supporting pre-cleaning (optional)**

Basically, we recommend pre-cleaning with ultrasound for all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the ILM-Scraper and make sure that it is completely moistened.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the ILM-Scraper afterwards with demineralized water for neutralization.

**AUTOMATIC CLEANING AND DISINFECTION**

As additional load only ophthalmic products are permitted.

Thermic disinfection A<sub>0</sub> value > 3000, 93 °C, 5 min; VARIO TD program.

The selection of suitable racks, charging trolleys for optimal rinsing and flushing of the medical devices should be done after consulting with the cleaning device manufacturer.

**Process**

- Place the ILM-Scraper in the rack (e.g. sieve baskets, trays):
  - Avoid rinsing shadows.

**DRYING**

Dry the ILM-Scraper with medical compressed air, if necessary.

**Control of pH value**

For ophthalmic instruments a batch-related routine control for alkaline residues is recommended.

- Apply the remaining humidity of the ILM-Scraper onto a pH test strip.
- Compare the pH test strip with the color scale:
  - pH must be neutral.

**CONTROL**

**Visual control**

- Check if the ILM-Scraper is clean:
  - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the ILM-Scraper is damaged:
  - Remove corroded or damaged ILM-Scraper (Fig. 2)
- Check if the ILM-Scraper is complete (Fig. 2).
- Check if the ILM-Scraper is dry.

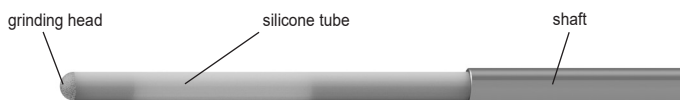
Fig. 2

**PACKAGING**

Use packaging materials and systems complying with the following standards: DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

**STERILIZATION**

Steam sterilization in fractionated prevacuum method: SAL 10<sup>-6</sup>, at least 3 min at 134 °C, 3 bar.

**STORAGE**

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

Damaged or open sterile barrier systems (SBS) are deemed non-sterile.

The storage time depends on the type of packaging and must be validated accordingly.

**TRANSPORT**

Transport must not have a negative effect on the characteristics of the sterile goods.

Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

**DISPOSAL**

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute ([www.rki.de](http://www.rki.de)).

National standards, regulations and/or restrictions must also be considered!

**GENERAL INFORMATION FOR PROCESSING**

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

**DETAILS OF VALIDATION**

**Summary of processes validated by GEUDER AG:**

The selection of the parameters refers to worst case conditions.

**Manual pre-cleaning**

**Pre-cleaning**

Using tap water (according TrinkwV [German Drinking Water Ordinance]), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

**Cleaning**

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H, after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

**Process chemicals (manual)**

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.

**Automatic cleaning and disinfection**

**Cleaning**

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

**Disinfection**

- Thermal high level disinfection A<sub>0</sub>-value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection A<sub>0</sub>-value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

**Process chemicals (automatic)**

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

**Steam sterilization**

- Program: half-cycle SAL 10<sup>-6</sup>, fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: type Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature (max. 25° C).
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective reprocessing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Actual version available at GEUDER AG.  
Subject to changes.

**Geuder**<sup>®</sup>

Herstellung ophtho-chirurgischer  
Instrumente und Gerätesysteme  
Manufacturer of Ophthalmic Surgical  
Instruments and Systems

GEUDER AG  
Hertzstrasse 4  
69126 Heidelberg GERMANY  
Phone : +49 6221 3066  
Fax: +49 6221 303122  
info@geuder.de  
www.geuder.de