**REF**

G-15930, G-15932, G-15940, G-15945, G-15946, G-15950, G-15951, G-15955, G-15956, G-15960, G-15965, G-15966, G-15970, G-15975, G-15990, G-15995, G-16000, G-16010, G-16020, G-16025, G-16030, G-16050, G-16055, G-17000, G-17001, G-17002, G-17003, G-17030, G-32128, G-32130, G-32135, G-33915, G-33965, G-33969, G-33971

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**EYE SPECULUM  
PRODUCT RELATED PROCESSING INSTRUCTIONS**

These product related processing instructions for semi-critical\* medical devices (MD) are exclusively valid for the above listed items by GEUDER AG. All medical devices that are provided in a non-sterile way must be reprocessed before each application.

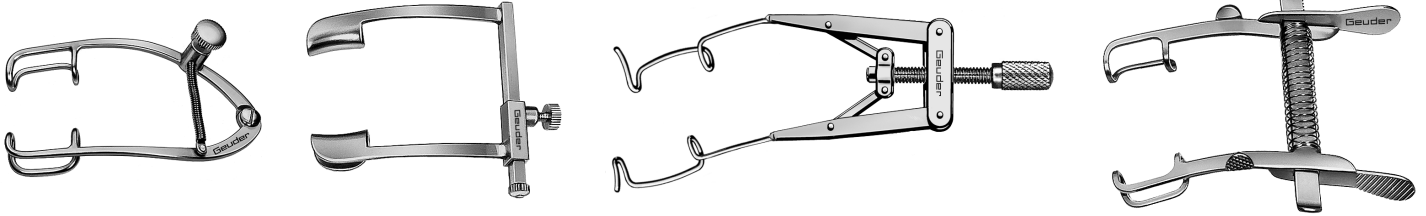
Fig. 1: example illustrations, not to scale

**OVERVIEW**

Cycle limit	Assembly/Disassembly	Accessories
NO	NO	NO
Pre-cleaning	Ultrasound	Automated processing
YES	YES	YES
Manual processing	Compressed air	Steam sterilization
YES	YES	YES

Tab. 1: Overview

**AUXILIARY EQUIPMENT**

<b>First treatment</b>	Soft tissue (e.g. compress)
<b>Pre-cleaning</b>	Soft plastic brush or steamer (steam jet, max. 2 bar), single use syringe (5 times with 10 ml each) or cleaning gun (water pressure gun, max. 2 bar) optionally: ultrasound bath 35-40 kHz
<b>Manual cleaning and disinfection</b>	Medical compressed air (oil-free compressed air, max. 2 bar)
<b>Drying</b>	Medical compressed air (oil-free compressed air, max. 2 bar)
<b>Checking pH value</b>	pH test strips
<b>Maintenance</b>	Instrument oil apt for sterilization, lint-free wipes

Tab. 2: Auxiliary Equipment

**FIRST TREATMENT AT PLACE OF USE**

- Remove macroscopically visible impurities (e.g. pharmaceuticals or corrosive solutions, etc.) during/ immediately after application with a compress soaked in sterile water.
- Place dry Eye Speculum into a suitable storage or transport system and close it.

**PREPARATION BEFORE CLEANING**

**Pre-cleaning**

- Rinse the Eye Speculum under running cold (< 40 °C) tap water.
- Remove visible, gross contamination using a soft plastic brush or a steamer.

**OR**

**Ultrasound cleaning as supporting pre-cleaning (optional)**

Basically, we recommend pre-cleaning with ultrasound for all complex or very dirty medical devices (e.g. instruments with a lumen, slide switch, joint, etc.). Observe the instructions or exposure time and concentration of the producer of the chemical as well as the instructions of the manufacturer of the ultrasound bath.

- Introduce the Eye Speculum and make sure that it is completely moistened:
  - Insert the Eye Speculum in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until the exposure time is over.
- Rinse the Eye Speculum afterwards with demineralized water for neutralization.

**MANUAL CLEANING AND DISINFECTION IN ULTRASOUND BATH**

**Manual cleaning in ultrasound bath**

- Introduce the Eye Speculum and make sure that they are completely moistened:
  - Insert the Eye Speculum in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the Eye Speculum into demineralized water for at least 2 minutes for neutralization:
  - Remove residues of debris with a soft plastic brush (without pressure).
- Rinse the Eye Speculum under running demineralized water.

**Manual disinfection in ultrasound bath**

- Introduce the Eye Speculum and make sure that it is completely moistened:
  - Insert the Eye Speculum in open conditions.
- Ultrasound treatment at 35-40 kHz, wait until exposure time is over.
- Afterwards, place the Eye Speculum into demineralized water for at least 2 minutes for neutralization.
- Rinse the Eye Speculum under running demineralized water.

**AUTOMATIC CLEANING AND DISINFECTION**

As additional load only ophthalmic products are permitted.

Thermic disinfection A<sub>0</sub> value > 3000, 93 °C, 5 min; VARIO TD program.

The selection of suitable racks, charging trolleys for optimal rinsing and flushing of the medical devices should be done after consulting with the cleaning device manufacturer.

**Process**

- Place the Eye Speculum in open condition in the rack (e.g. sieve baskets, trays):
  - Avoid rinsing shadows.

**DRYING**

Dry the medical devices with medical compressed air, if necessary.

**Control of pH value**

For ophthalmic instruments a batch-related routine control for alkaline residues is recommended.

- Apply the remaining humidity of the lumen instruments onto a pH test strip.
- Compare the pH test strip with the color scale:
  - pH must be neutral.

**CONTROL AND MAINTENANCE**

**Visual control**

- Check if the Eye Speculum is clean:
  - In case of residues, the cleaning and disinfection process must be repeated.
- Check if the Eye Speculum is damaged:
  - Remove corroded or damaged Eye Speculums.
- Check if the Eye Speculum is complete.
- Check if the Eye Speculum is dry.

**Maintenance**

- Care for the Eye Speculum with a suitable instrument oil:
  - Let the Eye Speculum cool down.
  - Apply the instrument oil selectively on the area that needs it (fig. 2).
  - Remove excess instrument oil, if any, with a lint-free wipe.
- Carry out a functional control:
  - Check if the Eye Speculum is complete.
  - Check if the moveable parts can be moved easily.
  - Check if the lock mechanisms work.

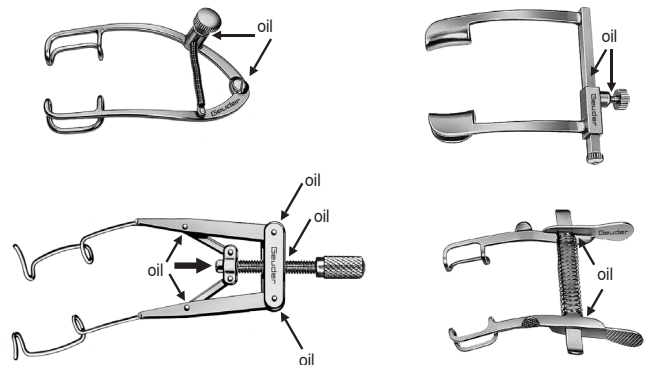
Fig. 2

**PACKAGING**

Use packaging materials and systems complying with the following standards: DIN EN (ANSI AAMI) ISO 11607 and DIN EN 868-8.

**STERILIZATION**

Steam sterilization in fractionated prevacuum method: SAL 10<sup>-6</sup>, at least 3 min at 134 °C, 3 bar.

**STORAGE**

Storage is at room temperature (max. 25 °C) in closed storage systems (dust- and light-protected also dry), protected from damage or mechanical influences, in a controlled environment.

- Damaged or open sterile barrier systems (SBS) are deemed non-sterile.
- The storage time depends on the type of packaging and must be validated accordingly.

\* Classification recommended by GEUDER AG, the responsibility lies with the operator.

## TRANSPORT

Transport must not have a negative effect on the characteristics of the sterile goods.

Returns to GEUDER AG: Please note that we only accept processed (cleaned, disinfected and sterilized) medical devices.

## DISPOSAL

The laws and provisions of each country must be complied with. Medical devices suspected to contain Creutzfeldt-Jakob Disease (CJD, vCJD): observe the recommendations of the Robert-Koch-Institute (www.rki.de).

National standards, regulations and/or restrictions must also be considered!

## GENERAL INFORMATION FOR PROCESSING

Detailed information for processing is provided in the manufacturer information for the processing of medical devices from GEUDER AG.

The life cycle of reusable medical devices ends, either due to material changes, material wear, damages, as well as functional changes or loss of function of the medical product, or after reaching the maximum number of reprocessing cycles in case of a cycle limitation as stated in the product related processing instructions for the product (see table overview).

## DETAILS OF VALIDATION

### Summary of processes validated by GEUDER AG:

The selection of the parameters refers to worst case conditions.

### Manual pre-cleaning, cleaning and disinfection

#### Pre-cleaning

Using tap water (according TrinkwV [German Drinking Water Ordinance]), cleaning gun or syringe, instrument cleaning brush with soft nylon bristles (15 mm).

#### Cleaning

- In the ultrasound bath, 0.5 %, 10 min/30-40 °C, 35 kHz, Bandelin RK 510 H, after ultrasound treatment the instruments are placed for 2 min in demineralized water and rinsed and/or operated.

#### Disinfection:

- 3.0 %, 15 min/20 °C, immersion bath, lumen were filled with disinfectant agent. After disinfection time rinse for 1 min with demineralized water.

#### Process chemicals (manual):

- Cleaning: Dr. Weigert: neodisher MediClean forte, demineralized water.
- Disinfection: Bode Chemie: Korsorex plus, demineralized water.

### Automatic cleaning and disinfection

#### Cleaning

- Program: shortened VARIO TD program, 0.2 %, 55 °C/10 min, final rinsing 3 min/demineralized water, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

#### Disinfection

- Thermal high level disinfection  $A_0$ -value > 3000, 93 °C/5 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.
- Thermal disinfection  $A_0$ -value = 600, 90 °C/1 min, Miele & Cie. KG: Washer-Disinfector type G7836, charging trolley: E450/1 accessories for MIC instruments, sieve basket with lid.

#### Process chemicals (automatic)

- Cleaning: Dr. Weigert: neodisher MediClean forte.
- Final rinsing: demineralized water.
- Disinfection: demineralized water.

### Steam sterilization

- Program: half-cycle SAL10<sup>-6</sup>, fractionated pre-vacuum procedure 3 times, 132 °C/1.5 min, MMM GmbH: type Selectomat PL/669-2 CL, sterilization pouches, drying time 20 min, cooling time: outside of the autoclave 30 min at room temperature (max. 25 °C).
- The maximum sterilization temperature is 138 °C.

An independent, accredited test laboratory proved the basic suitability of the medical devices for an effective reprocessing.

The above mentioned instructions were validated by the manufacturer of medical devices to be suitable for the processing/reprocessing of a medical product to be reused. The processing/reprocessing person(s) have the responsibility, that the actually applied procedure with the utilized equipment, materials and staff in the processing facility achieves the required result. Therefore the verification and/or validation and a routine monitoring of the procedure are indispensable.

Any serious incident that has occurred in relation to the medical device should be reported to the manufacturer and the competent authority.

Actual version available at GEUDER AG.

Subject to changes.